









Appendix 1

Cleaning PARO Seal

Things you need

Basin of cold water	 a basin of cold water	Fabric Detergent	 a fabric detergent
Fabric Softener	 a fabric softener	Fabric Pre-Wash	 a fabric pre-wash
Anti-Bac Febreeze	 anti-bac febreeze	Sponge (White Cloth or Sponge is best for PARO)	 a sponge cloth
Old Toothbrush	 an old toothbrush	Clean Towel	 a clean towel

General Method:

Pour a small amount of Fabric Detergent into the water



Dampen the Cloth (White cloth is best for PARO)



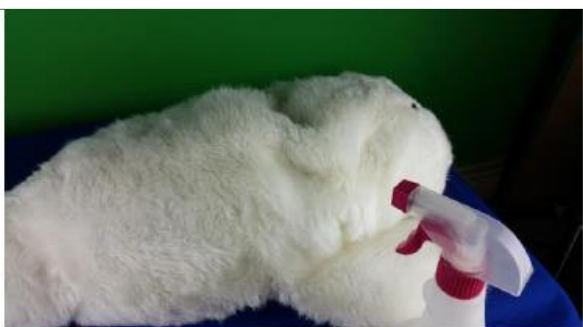

Squeeze out all excess water



Gently Rub your PARO's Fur






For stubborn dirt add the following steps:

<p>Spray the Fabric Pre-wash thoroughly on the stained area</p>	
<p>Gently Scrub it with a toothbrush</p>	

Time to rinse the PARO

<p>Fill up the water with Fabric Softener, damp the cloth and wring it out.</p>	 <p>Just fill up the water</p>
<p>Wipe the PARO</p>	

<p>Roll your PARO in towel</p>	
<p>Use a Soft Brush to give PARO a brush. (Brush the PARO before you dry PARO. This help to restore the sleek shiny look that the PARO originally had)</p>	
<p>Let PARO Dry</p>	
<p>Spray PARO with Anti-Bac Febreze</p>	